

THE CUTTING EDGE

KEIRSURGICAL

March/April 2010

CHEMICAL INDICATOR STRIPS FOR PERACETIC ACID STERILIZERS

SPSmedical has just introduced a Chemical Indicator Strip for use in the STERIS SYSTEM 1[®] Peracetic Acid (PA) sterilizer.

These chemical indicator strips are designed for qualitative detection of peracetic acid. The peracetic acid oxidizes the indicator dye, leaving the chemical indicator pad colourless.



PA Chemical Indicator Strips (PAS-050)



SporView PA Culture Set (PCS-025)

The chemical indicators works in three basic steps:

1. Place the chemical indicator strip into the STERIS SYSTEM 1[®] Processor with the plastic clip provided
2. Run a standard processing cycle outlined by the STERIS SYSTEM 1[®] Operator Manual
3. Extract the strip and compare the colour of the chemical indicator pad to the "COMPLETE" colour block on the bottle label.

The colour of the chemical indicator pad will change from the colour of the "START" block to the colour of the "COMPLETE" block or lighter depending if the active ingredient is greater than 1500ppm.

The PA Chemical Indicators Strips are recommended to be used in conjunction with the SPSmedical SporView PA Culture Set[®], the fastest biological indicator results in the industry.

SINGLE-USE INSUFFLATION BELLOWS



Box of 25 Single Use Bellows (Code: 69.1890/25)

Single-Use Insufflation Bellows reduce the risk of cross-infection in sigmoidoscopy and eliminate the need for expensive hydrophobic filters. Keir Surgical's bellows are latex free, compatible with most sigmoidoscopes, sold in boxes of 25 with each item individually wrapped.

Contact your sales representative for more information about the Single-Use Bellows.

KS

KEIRSURGICAL

126-408 East Kent Avenue South
Vancouver, BC V5X 2X7

T: 800.663.4525 | F: 604.261.9549

info@keirsurgical.com | www.keirsurgical.com

Please contact your Keir Surgical Sales Representative for more information.

THE CUTTING EDGE

KEIRSURGICAL

March/April 2010

SINGLE-USE PROCTOSCOPES

Also offered by Keir Surgical are Single-Use Proctoscopes and Single-Use Light Stems. The one-time use instruments help reduce the risk of cross-infection.

Keir Surgical carries both adult and pediatric proctoscopes. The adult proctoscope measures 70 mm in length and 19 mm in diameter whereas the pediatric proctoscope measure 70 mm in length and 9 mm in diameter.

The Single-Use Light Stem, which comes with either a Storz or Welch Allyn light cable fitting, provides superior distal tip illumination for proctology, allowing easier access for ligating or injecting of haemorrhoids.

Each of the above products is latex-free in individually wrapped.



Adult Proctoscope, box/25 (69.0365/025)
Pediatric Proctoscope, box/10 (69.0366)

Light Stem, Storz Fitting, box/25 (69.0635/025)
Light Stem, Welch Allyn Fitting, box/25 (69.0638/025)

Did You Know?

- Keir Surgical launched its new eStore on February 5, 2010. Surgical and Post Surgical products are now available for online browsing and purchase. Check out www.keirsurgical.com/eStore for more details.
- On Saturday March 27, Keir Surgical is hosting an educational seminar in partnership with SPSmedical and RUDOLF MEDICAL. Topics covered at the all-day event include: instrument manufacturing & maintenance and sterilization & infection control. Leading the workshops are Charles Hughes (SPSmedical) and Daniel Favre (RUDOLF MEDICAL).

KS

KEIRSURGICAL

126-408 East Kent Avenue South
Vancouver, BC V5X 2X7

T: 800.663.4525 | F: 604.261.9549

info@keirsurgical.com | www.keirsurgical.com

Please contact your Keir Surgical Sales Representative for more information.